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Laryngeal Electrodes

Intubation Instructions

Single Channel Laryngeal electrodes for I.O.M.
with 75 cm cables and T.P. DIN 42802 connectors

ENGLISH

INTENDED USE

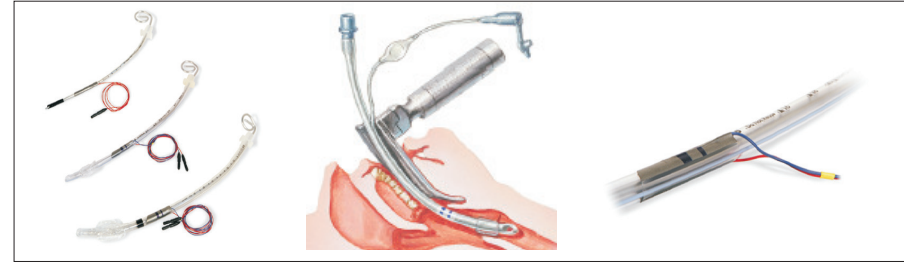
The Laryngeal Surface Electrode is intended to be used as disposable, self-adhesive electrode attached to an endotracheal tube and positioned for continuous EMG monitoring of the larynx during surgical procedures. It is intended for use only by a licensed physician and in conjunction with a commercially available, medical grade electromyographic monitor.

CLINICAL APPLICATIONS

To be used for intraoperative monitoring during Surgery operations for:
Anterior Cervical Fusion- Carotid Endarterectomy-Thyroidectomy- Skull Base Procedures- Craniotomy

PACKAGING

Each electrode is packed in a pouch . One box contains 10 Sterile electrodes of the same type

Laryngeal Electrode Intubation Instruction

Reliable performance of LSEXXXMCA075 electrodes requires proper positioning. Please follow these instructions carefully and avoid using long acting paralytics

- 1) Choose any E.T. tube and the appropriate electrode based on the E.T. tube size chart below

Electrode Type	LSE4246MCA075	LSE3446MCA075
E.T. Tube size	7.5 - 10.0	6.0 - 7.0

- Determine the location where the vocal cords will be in contact with the ET tube after intubation. The middle of the electrode will be applied at this location with the wires directed toward the mouth
- Turn over and straighten the ET tube to expose what will be the posterior side when intubated. Use of a straightened stylet facilitates hands-free straightening of the ET tube
- Remove the paper backing from the electrode (3M 300LSE) and apply approximately 1 cm above the balloon and with the wires directed toward the mouth. Align electrode's midline with the midline of the posterior portion of the tube. Press electrode down, first wrapping the larger flap toward the top (anterior) side of the tube. Then wrap the smaller flap until they overlap.
- A small amount of lubricant may be applied to the electrode. Insert the E.T. Tube under direct vision that each vocal cord is touching its respective electrode plates and rests between the two blue positioning stripes.
- Note the depth number on the ET tube against the maxillary central incisors before any further positioning of the patient.
- After final positioning of patient, align ET tube in the middle of the pharynx behind the tongue. The posterior portion of the ET tube should be directly opposite the central maxillary incisor gap at the depth number noted after initial positioning.
- Tape the ET tube securely with 2 pieces of tape by wrapping each piece first around the tube and then securing to the upper lip.
- Tightly secure the breathing circuits so the ET tube will not rotate or be displaced and then verify final electrode position by laryngoscopy with a #3 Miller Blade.
- Attach the red and blue electrode lead wires to the + and - EMG terminals and apply an EMG ground

STERILE EO

Use Only

Latex free

Do not resterilize

Do not use if package
is damaged

Caution: This single-use product is not designed for re-use. Attempts to clean and re-use this single-use device expose patients to cross contamination.
Caution: A normal laryngeal EMG recording indicates an intact neuromuscular axis, but does not guarantee vocal fold mobility or guarantee muscle function in a partially denervated or deconditioned muscle.

Spes Medica S.r.l.
Via Europa-Zona Industriale
84091 Battipaglia (SA) Italy
Tel: +39 (0)828 614191
Fax: +39 (0)828 341788
www.spesmedica.com, info@spesmedica.com

Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a practitioner licensed by the law of the State in which he/she practices

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ITALIANO


Laryngeal Electrodes

Connection Set up
Single Channel Laryngeal electrodes for I.O.M.
with 75 cm cables and T.P. DIN 42802 connectors

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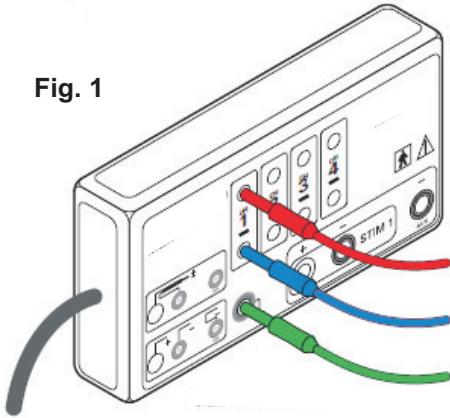Fig. 2



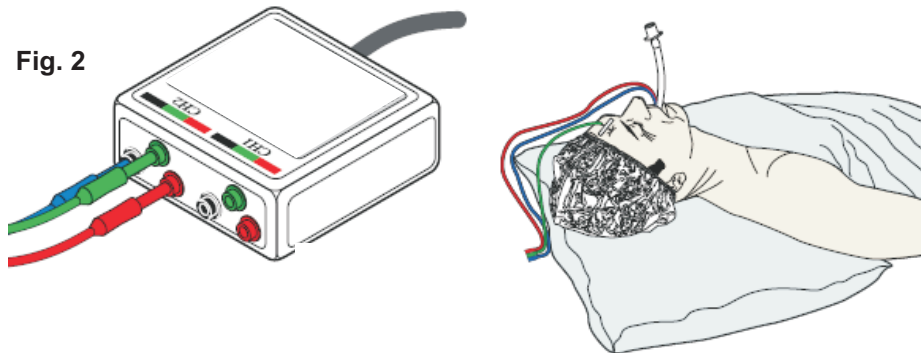
Fig. 1

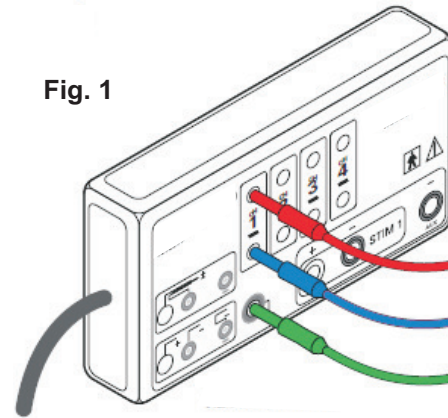CONNECTING THE LARYNGEAL SINGLE CHANNEL ELECTRODE TO THE SURGICAL NERVE MONITOR

- 1 - Secure the Patient Interface within 1,5 m of the patient's head;
- 2 - Using the two channel electrode **LSEXXXMCA075**.
Plug the red and blue electrode lead wire connectors into the appropriate ports as shown in figures 1 and 2;
- 3 - Plug the green ground electrode lead wire connector into the appropriate port(s) as shown in figures 1 and 2;
- 4 - Set up the remaining components and setting as directed in the manufacturer's instruction of the surgical nerve monitor chosen.

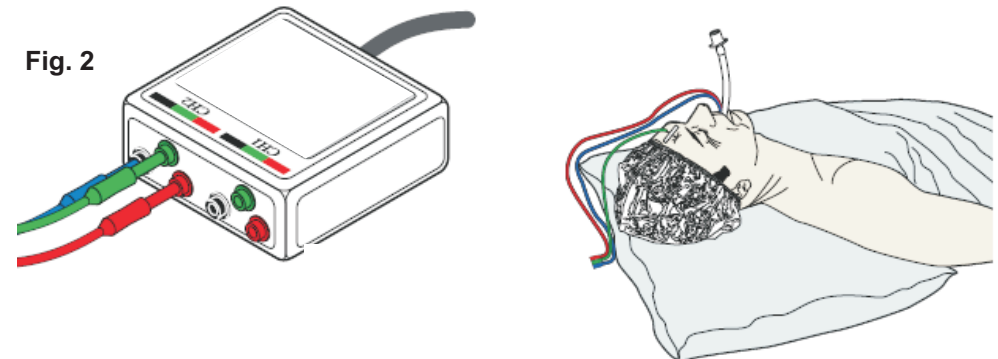Fig. 2

Spes Medica S.r.l.
Via Europa-Zona Industriale
84091 Battipaglia (SA) Italy
Tel: +39 (0)828 614191
Fax: +39 (0)828 341788

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