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<b>Intended use</b>
The Needle Electrodes are intended for use for the recording of biopotential signals with recording, monitoring and stimulation equipment for neurophysiology, examples include: Electromyography (EMG), Electroencephalography (EEG), intraoperative monitoring (IOM) and evoked potentials (EP).
<b>Instructions for use</b>
<ol style="list-style-type: none"><li>Do not use the selected needle if the pouch is damaged or if the product has expired;</li> <li>Open the sterile pouch and remove the needle.</li> <li>Connect the wire / wires to the appropriate connectors.</li> <li>Remove the cover of the needle.</li> <li>Insert the needle into the area to be tested.</li> <li>Remove the hypodermic needle taking care that the wire / wires remain inserted into the patient</li> <li>Dispose the needle according to standard protocols.</li> <li>After use, dispose the wires according to the standard protocols.</li></ol>

<b>Warnings</b>
<ul style="list-style-type: none"><li>The electrode is for single use only.</li> <li>This single-use product is not designed for re-use. Attempts to clean and re-use this single-use device expose patients and operators to a risk of cross contamination.</li> <li>Do not re-sterilize.</li> <li>To avoid contamination of the device and therefore of the patient, the user must wear disposable gloves</li> <li>Only to be used by the clinical specialist</li> <li>It is necessary for the user and/or patient to report any serious incident occurring, related to the device, to the manufacturer and the competent authority of the member state where the user and/or patient is established.</li> <li>The environment of use is a hospital, specifically an operating room. The device is intended for both adult and pediatric patient populations. The device does not have limitations of use related to age (from birth onward), gender, or weight. The clinical specialist will identify the correct size for the clinical need.</li> <li>There are no contraindications or warnings for pregnant or lactating women.</li></ul>

<span></span>	Complies with the Regulation (EU) 2017/745	<span></span>	<span></span>
<span></span>	Swiss authorized representative	<span></span>	<span></span>
<span></span>	Medical device	<span></span>	Unique device identifier
<span></span>	Catalogue number	<span></span>	Lot number
<span></span>	Single Use Only	<span></span>	Do not resterilize
<span></span>	Consult instruction for use	<span></span>	Do not use if package is damaged
<span></span>	Warnings and Caution associated with this device.	<span></span>	Expire by
<span></span>	Ethylene Oxide Sterilization with single sterile barrier system	<span></span>	Ethylene Oxide Sterilization with single sterile barrier system with protective packaging outside
<span></span>	Manufacturer	<span></span>	

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<b>CE</b> <b>0051</b>	Regulation (EU) 2017/745 in accordance with
<b>CH</b> <b>REP</b>	Swiss authorised representative
<b>MD</b>	Medical device
<b>UDI</b>	Individual device identification number (UDI)
<b>REF</b>	Product number
<b>LOT</b>	Lot number
	Reuse is prohibited
	Reuse is prohibited
	Read the user manual
	Do not reuse if the packaging is damaged
	Warnings
	Use-by date
	Sterilized once in a single-use sterile pack
	Ethylene oxide sterilized with a single-use sterile barrier or airtight packaging with a disinfectant wipe
	Manufacturer

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<b>CE</b> <b>0051</b>	Regulation (EU) 2017/745 in accordance with
<b>CH</b> <b>REP</b>	Swiss authorised representative
<b>MD</b>	Medical device
<b>UDI</b>	Individual device identification number (UDI)
<b>REF</b>	Product code
<b>LOT</b>	Lot number
	Reuse is prohibited
	Reuse is prohibited
	Read the user manual
	Do not reuse if the packaging is damaged
	Warnings
	Use-by date
	Sterilized once in a single-use sterile pack
	Ethylene oxide sterilized with a single-use sterile barrier or airtight packaging with a disinfectant wipe
	Manufacturer

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