Myoline Single Fiber *Instruction For Use*

ENGLISH

EMG DISPOSABLE SINGLE FIBER EMG BIPOLAR NEEDLE

A technologically advanced disposable bipolar needle electrode designed for patient comfort along with top reliability and safety for the user.

CLINICAL APPLICATION


To be used for Single Fiber Electromyography tests (SFEMG)

PACKAGING

Each pack contains one ready for use sterile needle, to be used for one patient only, not resterilizable. Each box contains 5 needles.

TECHNICAL FEATURES

- 1- Moulded needle with ergonomic handgrip
- 2- Special tip sharpening for patient and user friendly introduction
- 3- Recording area marker
- 4- Press-on tongues to automatically disconnect needle after introduction into hospital disposal container
- 5- Easy to handle protective sheath in rigid material
- 6- ETO sterilizable light and flexible leadwire, diameter 1,2 mm, with 5-pole DIN 45322 connector. (Provided separately)
- 7- Moulded connections on both ends
Entirely shielded handgrip connected with the grounding of the device

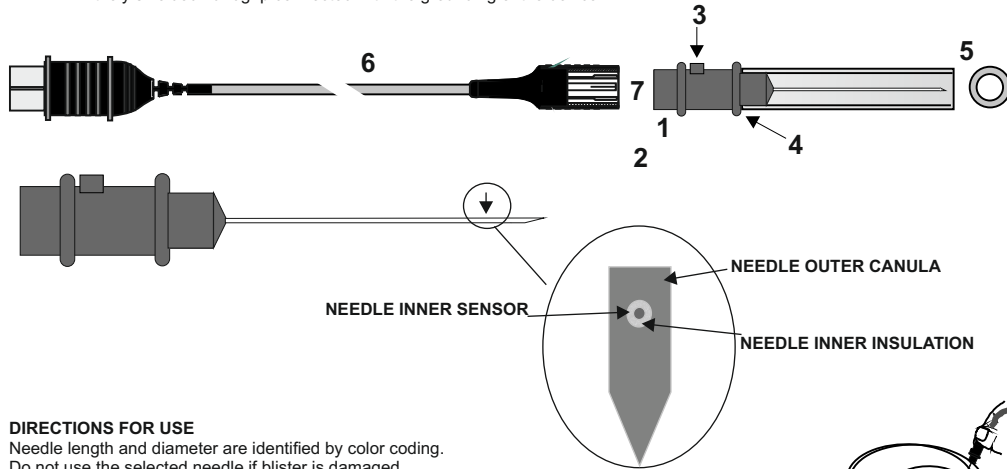DIRECTIONS FOR USE

Needle length and diameter are identified by color coding. Do not use the selected needle if blister is damaged.

Open the pack following the instructions on the sterile pouch:

- 1) Remove needle from sterile pouch
- 2) Connect needle to connector
- 3) Remove the needle sheath
- 4) Introduce needle into area to be tested taking into account position of recording area marker.
- 5) After test dispose of needle using the same procedure as, and a specific container for subdermal needle, or alternatively enclose needle back into sheath before disposal.

	Complies with the Regulation (EU) 2017/745		Consult instruction for use		Do not use if package is damaged
	In the United State, Federal law restricts this device to sale by or on the order of a physician practitioner.		Warnings and Cautions associated with this device.		Expire by
	Swiss authorized representative		Ethylene Oxide Sterilization with single sterile barrier system		Ethylene Oxide Sterilization with single sterile barrier system with protective packaging outside
	Medical Device		Unique device identifier		Manufacturer
	Catalogue number		Lot number		
	Single Use Only		Do not resterilize		

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