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Disposable Adhesive Surface Electrodes
ENGLISH

Intended use

Disposable Adhesive Surface Electrodes are intended for use along with electromedical devices for the recording and /or the electrical stimulation.

Instructions for use

- Before placing the electrode, clean and degrease the skin on the area of the electrode placement scrubbing it gently with a skin prep creme, in order to reduce the skin impedance.
- Clean the patient’s skin with a damp cotton ball.
- Remove the transparent backing from the electrode and place the electrode on the prepared area.
- To assure good contact apply pressure in the center of the electrode and move to the edges.
- At the end of the exam remove the electrode by pulling directly on it.

Waste Disposal

Waste coming from hospitals must be disposed of in accordance with regulations in force.

⚠ Warnings

- The electrodes are non-sterile and for single patient only.
- Do not pull on the cable, when electrode is applied.
- Clean any remaining of gel with damp cotton ball.
- Discard Electrodes after use to avoid any cross-contamination.

CE	Complies with the Regulation (EU) 2017/745
Rx Only	In the United State, Federal law restricts this device to sale by or on the order of a physician practitioner.
REF	Catalogue number
LOT	Lot number
MD	Medical Device
UDI	Unique device identifier
	Single use only
	Keep away from heat
	Temperature limitation
	Expire by
	Consult instruction for use
	Manufacturer

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