www.spesmedica.com
16135 Genova (GE), Italia
VIA Boccia 21
Spes Medical s.p.a.

Disposable Urethral Electrode

Intended use
Embolization catheters to be used for the detection of biopotentials signals of pelvic floor and/or urethral sphincter muscle with recording, monitoring and stimulation equipment.

Instructions for use

- Check the catheter's smaller size (French (Fr.): 1 French is roughly equivalent to 0.33 mm in diameter, the smaller the number the smaller the catheter). Suitable sizes for children are from 8Fr. to 10Fr. for adults from 14Fr. to 20Fr.
- Before starting the procedure, clean the genital area with a mild soap, warm water and then patted dry.
- Do not use the urethral electrode if the pouch is damaged or if the product has expired.
- Open the sterile pouch and remove urethral electrode.
- Lubricate the catheter with a water-based lubricant, avoiding the contact between the lubricant and the electrode recording and stimulation area.
- Open the sterile pouch and the urethra as a normal catheter getting a stable interface between the electrode recording area and the external sphincter muscle.
- To maintain catheter position in place in bladder, inflate the balloon with a saline solution using a syringe. The following table inflation values:

Fr	Balloon size
6	1,5 ml
8	3 ml
10	5 ml
12-20	10 ml

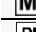
- Create a gentle outward traction on the catheter maintaining the correct placement of the balloons against the internal urethral sphincter.
- Connect the urethral electrode cable with the equipment.
- Once the catheter is over, remove completely the saline solution from the balloon using a syringe without needle. Then gently remove the catheter.
- The device should be removed within 10 minutes at the end of the surgery as determined by a physician or other suitably qualified personnel who are familiar with this device.
- The stimulation electrical pulses should have a duration of ms and an intensity not exceeding 60-100 mA.
- During the catheter lubrication, avoiding the contact between the lubricant and the electrode recording and stimulation area.
- Only to be used by the clinical specialist.
- It is necessary for the user and/or patient to report any serious incident occurring related to the device, to the manufacturer and the competent authority of the member state where the user and/or patient is established.
- The environment of use is a hospital, specifically an operating room.
- Zakres zastosowania dla obu płci dla pediatrycznej populacji. The device does not have limitations of use related to age (from birth onward), gender, or weight. The clinical specialist will identify the correct size for the clinical need. There are no contraindications or warnings for pregnant or lactating women.

Warnings

- The urethral electrode is for single patient only. It is not designed for re-use. Use of the device and/or the single-use device expose patients and operators to a risk of cross contamination.
- Gloves must be worn by the operator during the catheterization to protect himself from infection and to avoid infection in the patient.
- Do not re-sterilize.
- Do not handle the catheter with sharp instruments.
- The device should be removed within 10 minutes at the end of the surgery as determined by a physician or other suitably qualified personnel who are familiar with this device.
- The stimulation electrical pulses should have a duration of ms and an intensity not exceeding 60-100 mA.
- During the catheter lubrication, avoiding the contact between the lubricant and the electrode recording and stimulation area.
- Only to be used by the clinical specialist.
- It is necessary for the user and/or patient to report any serious incident occurring related to the device, to the manufacturer and the competent authority of the member state where the user and/or patient is established.
- The environment of use is a hospital, specifically an operating room.
- Zakres zastosowania dla obu płci dla pediatrycznej populacji. The device does not have limitations of use related to age (from birth onward), gender, or weight. The clinical specialist will identify the correct size for the clinical need. There are no contraindications or warnings for pregnant or lactating women.

Contraindications and warnings for pregnant or lactating women

Complies with the Regulation (EU) 2017/745
In the United States, Federal law restricts this device to sale by or on the order of a physician practitioner.

Rx Only	Complies with the Regulation (EU) 2017/745	UDI	Unique device identifier
CH REP	Swiss authorized representative	UDI	Unique device identifier
MD	Medical Device	UDI	Unique device identifier
REF	Catalogue number	LOT	Lot number
	Single Use Only		Do not re-sterilize
	Consult instruction for use		Do not use if package is damaged
	Warnings and Cautions associated with this device		Epilepsy
	Ethylene Oxide Sterilization with single sterile barrier system		Ethylene Oxide Sterilization with single sterile barrier system packaging outside
	Manufacturer		Manufacturer

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