SpS Medical S.p.A. Via Biccarti 21 16153 Genova (Italy) Tel. +39 010 3072345 Fax. +39 010 3072345 Web: www.spesmedica.com

SAC2 – Electrode Cream

Instructions for use
<ol style="list-style-type: none">Prepare the patient’s skin to reduce the skin impedance: gently scrub the skin using an abrasive skin prep paste; Apply a small amount of SAC2 on the skin, place the electrode on the scalp and then apply SAC2 till submerge the electrode; Place a small gauge pad over the electrode to help holding the electrode in place; To remove the cream use a wet cloth.

⚠ Warnings

- SAC2 is for single patient and single use only. Attempts to re-use this single-use product expose patients and operators to a risk of cross-contamination.
- The electroconductive media should not be reused.
- Use only on external areas under professional guidance.
- SAC2 must be applied only to a clean electrode to be placed over normal, intact, clean, healthy skin.
- The electroconductive media should not come into contact with open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins.
- The electrical performance characteristics of the electroconductive media may affect the safety and effectiveness of recording.
- Do not use if the tube is damaged.
- Keep away from eyes.
- Keep away from sunlight.
- Use SAC2 before date on end of tube.
- Maximum duration of use: 7 days.
- Adverse Reactions:**
- Patients should stop using the device and should consult with a physician if they experience adverse reactions from the device.

CE	Complies with the Regulation (EU) 2017/745
Rx Only	In the United State, Federal law restricts this device to sale by or on the order of a physician practitioner
REF	Catalogue number  Consult instruction for use
LOT	Lot number  Warnings and Cautions
MD	Medical Device  Expire by
 	Keep away from sunlight  Do not use if package is damaged
	Manufacturer

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SAC2 – Electrode Cream

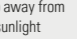
ENGLISH

Intended use
<p>SAC2 is intended for use in clinical and research EEG/EP recordings from humans. It is used with external electrodes as the conductor between the scalp and recessed electrodes to reduce impedance between the electrode surface and the skin.</p>

- Prepare the patient’s skin to reduce the skin impedance: gently scrub the skin using an abrasive skin prep paste;
- Apply a small amount of SAC2 on the skin, place the electrode on the scalp and then apply SAC2 till submerge the electrode;
- Place a small gauge pad over the electrode to help holding the electrode in place;
- To remove the cream use a wet cloth.

⚠ Warnings

- SAC2 is for single patient and single use only. Attempts to re-use this single-use product expose patients and operators to a risk of cross-contamination.
- The electroconductive media should not be reused.
- Use only on external areas under professional guidance.
- SAC2 must be applied only to a clean electrode to be placed over normal, intact, clean, healthy skin.
- The electroconductive media should not come into contact with open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins.
- The electrical performance characteristics of the electroconductive media may affect the safety and effectiveness of recording.
- Do not use if the tube is damaged.
- Keep away from eyes.
- Keep away from sunlight.
- Use SAC2 before date on end of tube.
- Maximum duration of use: 7 days.
- Adverse Reactions:**
- Patients should stop using the device and should consult with a physician if they experience adverse reactions from the device.

CE	Complies with the Regulation (EU) 2017/745
Rx Only	In the United State, Federal law restricts this device to sale by or on the order of a physician practitioner
REF	Catalogue number  Consult instruction for use
LOT	Lot number  Warnings and Cautions
MD	Medical Device  Expire by
 	Keep away from sunlight  Do not use if package is damaged
	Manufacturer

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