Neurgel – Conductive Gel

ENGLISH

Intended use

Neurgel is intended for use in clinical and research EEG/EP/EMG recordings from humans. It is used with external electrodes as the conductor between the scalp and recessed electrodes to reduce impedance between the electrode surface and the skin.

Instructions for use

- Prepare the patient’s skin to reduce the skin impedance: gently scrub the skin using an abrasive skin prep paste;
- Place the electrode on the scalp and then insert a small amount of Neurgel by a syringe with blunted needle;
- After the exam, remove the gel using a wet cloth.

Warnings

- Neurgel is for single patient and single use only. Attempts to re-use this single-use product expose patients and operators to a risk of cross-contamination.
- The electroconductive media should not be reused.
- Use only on external areas under professional guidance.
- Neurgel should be applied only to a clean electrode that is placed over normal, intact, clean, healthy skin.
- The electroconductive media should not come into contact with open wounds or rashes, or over swollen, red, infected, or inflamed areas, or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins
- The electrical performance characteristics of the electroconductive media may affect the safety and effectiveness of recording
- Do not use if the packaging is damaged.
- Keep away from eyes.
- keep away from sunlight.
- Use Neurgel before the expiration date.
- Adverse Reactions**
- Patients should stop using the device and should consult with a physician if they experience adverse reactions from the device

CE	Complies with the Regulation (EU) 2017/745		
CH REP	Swiss authorized representative		
REF	Catalogue number		Consult instruction for use
LOT	Lot number		Warnings and Cautions
MD	Medical Device		Expire by
	Keep away from sunlight		Temperature limitation
UDI	Unique device identifier		Manufacturer

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