SafePrep – Abrasive Gel
Instruccions for use

Intended use
SafePrep is intended for use in clinical and research EEG/EP/EMG recordings from humans. It is used with external electrodes to prep the skin to reduce the cutaneous impedance.

Instructions for use

- Place a small amount of SafePrep on a gauze.
- Gently scrub the skin area to prep with the gauze with the Safe Prep by circular movements
- Remove the excess of gel.

Warnings

- SafePrep is for single patient and single use only. Attempts to re-use this single-use product expose patients and operators to a risk of cross-contamination.
- The product should not be reused.
- Use only on external areas under professional guidance.
- SafePrep should be applied only over normal, intact, clean, healthy skin.
- SafePrep should not come into contact with open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins
- Do not use if the packaging is damaged.
- Keep away from eyes.
- Keep away from sunlight.
- Use SafePrep before the expiration date.
- Adverse Reactions
- Patients should stop using the device and should consult with a physician if they experience adverse reactions from the device

	Complies with the Regulation (EU) 2017/745
	Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a practitioner licensed by the law of the State in which he/she practices
	Swiss authorized representative
	Catalogue number
	Consult instruction for use
	Lot number
	Warnings and Cautions
	Medical Device
	Expire by
	Keep away from sunlight
	Temperature limitation
	Unique device identifier
	Manufacturer

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